


2019-nCoV

Antigen Rapid Test Kit

(Colloidal Gold Immunochromatography)

CE

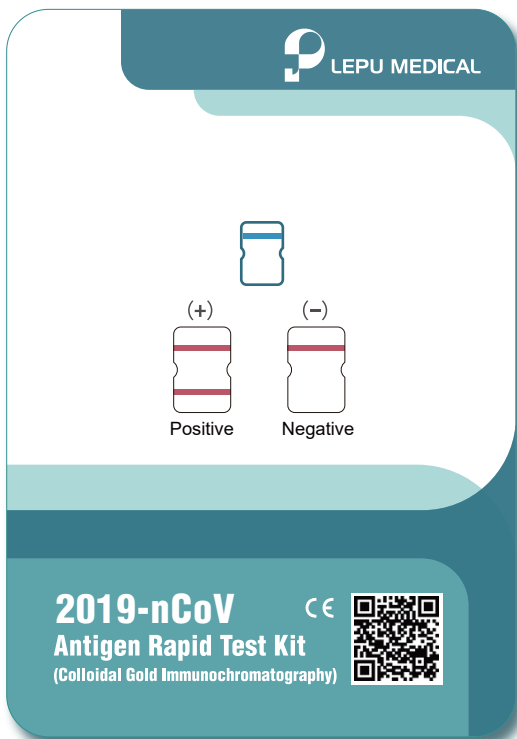

- ① Non-invasive
- ② Simple to use
- ③ Convenient, no devices required
- ④ Rapid, get result in 15 minutes
- ⑤ Stable, with high accuracy
- ⑥ Inexpensive, cost-efficiency



Hospital



Test Site



Airport



Station



Hotel




Corporation




Mass
Screening




Instruction

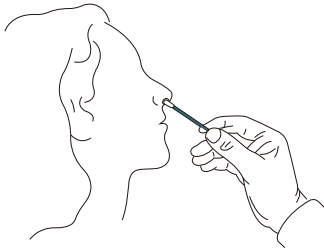


Step 1: Use swab to collect sample.

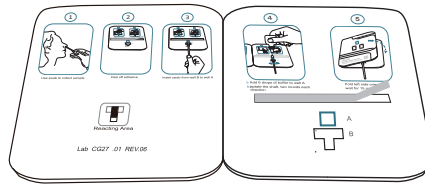


Step 2: Peel off adhesive.

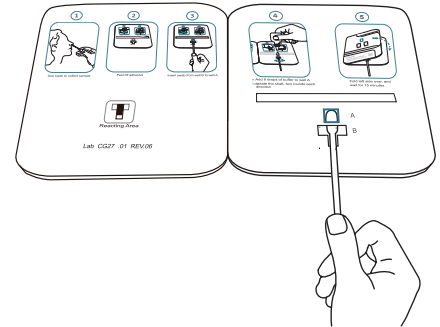

Step 3: Insert swab from well B to well A.

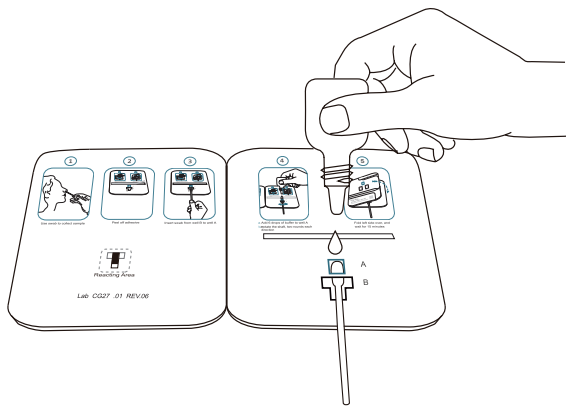

Step 4: a. Add 6 drops of buffer to well A
b. Rotate the shaft, two rounds each direction.

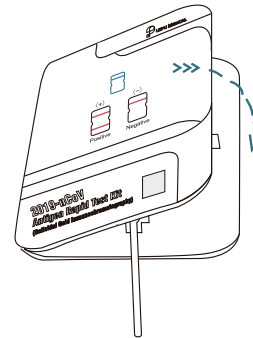

Step 5: Fold left side over, and wait for 15 minutes.



Result Interpretation


Positive

(+)




Negative

(-)

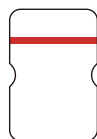

Invalid

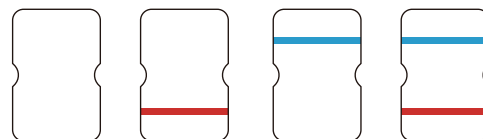

BEIJING LEPU MEDICAL TECHNOLOGY CO., LTD.

Building 7-1 No.37 Chaoqian Road, Changping District, Beijing, 102200, P.R.China

Tel: +86-15120061834

Email: yedda.sui@lepu-medical.com

Web: en.lepumedical.com